

Cog


Lifting and Holding

6D & Bi-directional Cog threads make a strong lifting effect. It became safer by using W-type cannula.



	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	17G	100	WS WL	3&4	160	12
	18G	100		2	160	
PCL	19G	100		1	160	

☆☆ SUPER SIZE Cog

USP
3&4

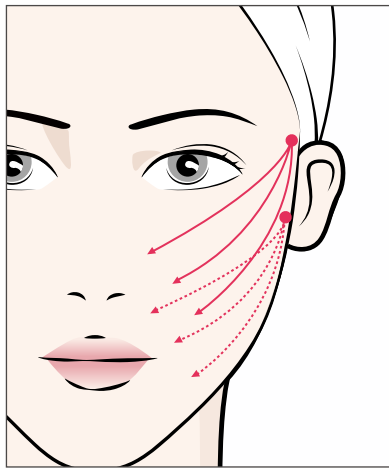

- 1/ After a punching, Insert into the SMAS.
- 2/ Insert them to the desired point.
- 3/ Remove a fixing straw in the handle
- 4/ While pulling the product about 1cm, push the thread into the skin to fix it.
- 5/ Remove cannula while pressing thread in the skin by hand.
- 6/ Lift up properly. Balance and cut the thread.

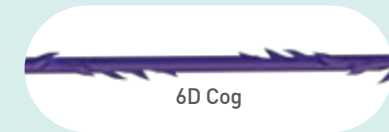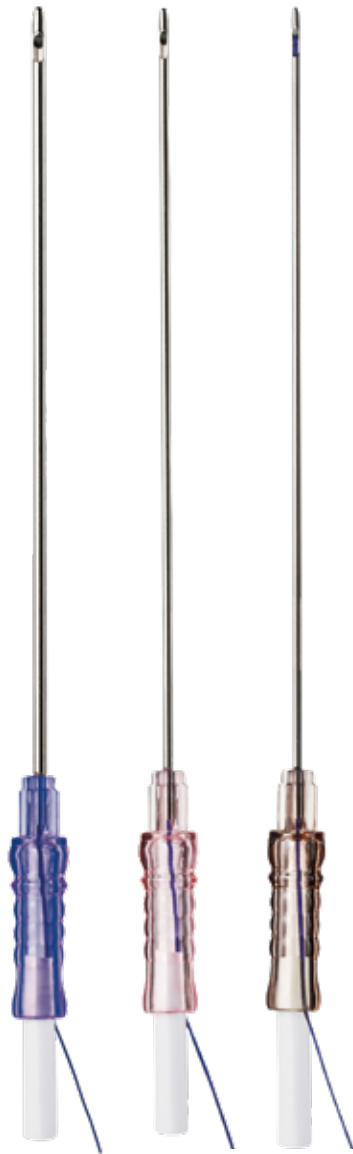


Cog



Lifting and Holding

6D & Bi-directional Cog threads make a strong lifting effect. It became easier by using L-type cannula.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	18G	100	LA	2	200	12
	19G	100		1	200	
	19G	60		1	140	
	21G	90		2-0	180	
	21G	60		2-0	140	
	23G	90		3-0	180	
PCL	23G	60		3-0	140	
	18G	100		2	200	
	19G	100		1	200	

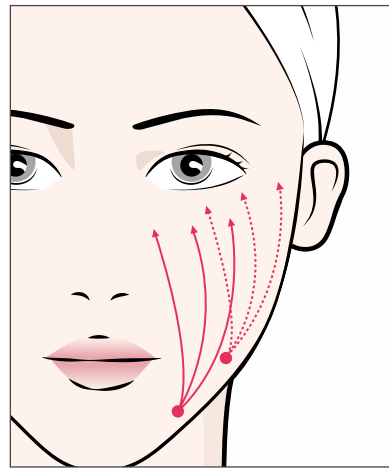

- 1/ After a punching, Insert into the SMAS.
- 2/ Insert them to the desired point.
- 3/ Remove cannula while pressing thread in the skin by hand.
- 4/ Lift up properly.
- 5/ Balance and cut the thread.

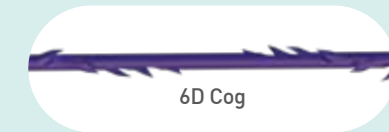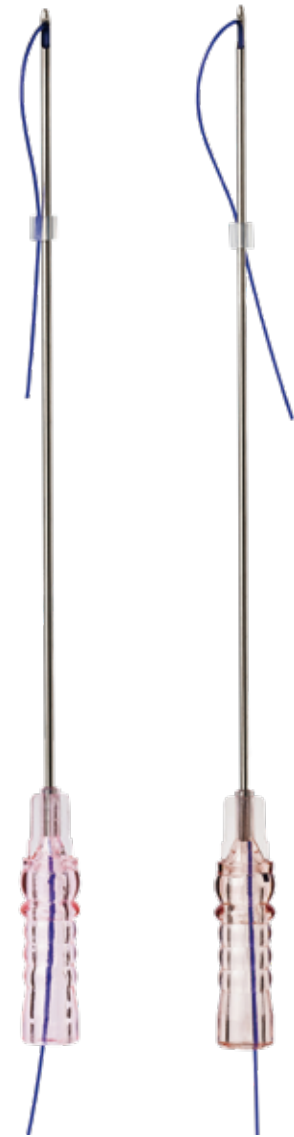



Molding Cog

Lifting and Holding


The Strongest molding thread to lift & hold with USP3&4.
3rd generation molding thread: Without heat treatment.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	18G	100	WS	3&4	160	12
	19G	100		3&4	160	

☆☆ Strong Lifting Effect

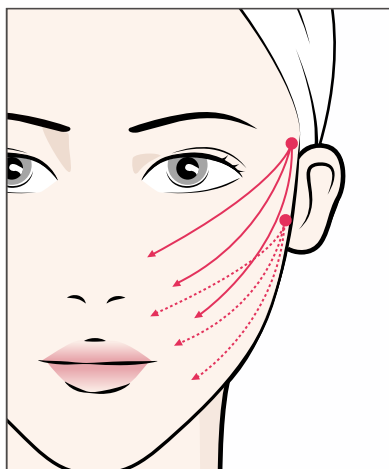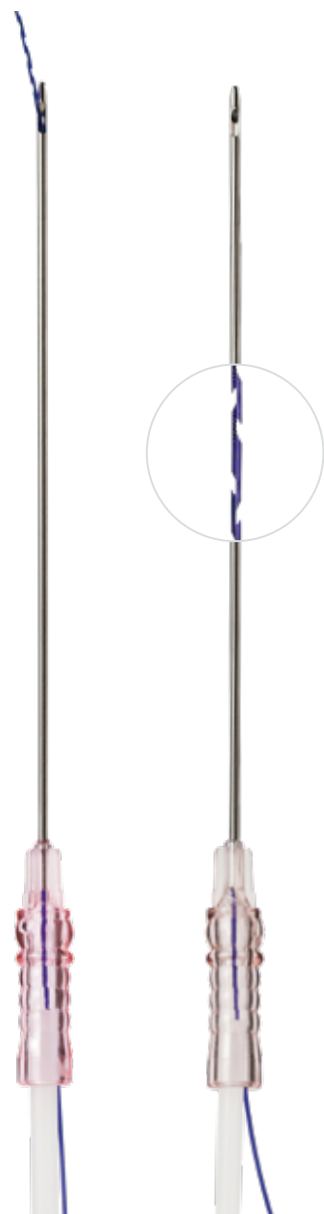

- 1/ After a punching, Insert into the SMAS.
- 2/ Insert them to the desired point.
- 3/ Remove a fixing straw in the handle
- 4/ While pulling the product about 1cm, push the thread into the skin to fix it.
- 5/ Remove cannula while pressing thread in the skin by hand.
- 6/ Lift up properly. Balance and cut the thread.

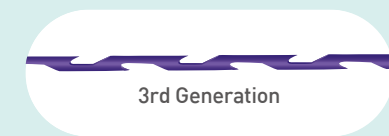


Molding Cog

Lifting and Holding


The Strongest molding thread to lift & hold with USP3&4.
3rd generation molding thread: Without heat treatment.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PCL	18G	100	WL	3&4	175	12
	19G	100		3&4	175	

☆☆ Strong Lifting Effect

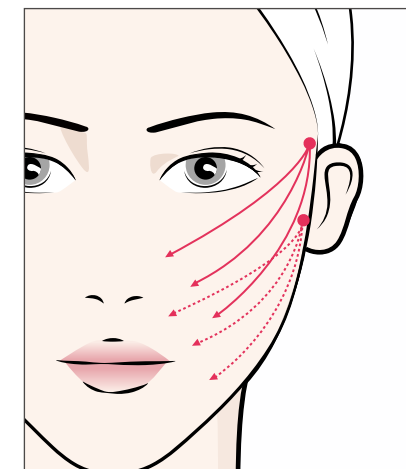

- 1/ After a punching, Insert into the SMAS.
- 2/ Insert them to the desired point.
- 3/ Remove cannula while pressing thread in the skin by hand.
- 4/ Lift up properly.
- 5/ Balance and cut the thread.

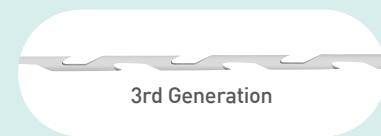


Double Cog


Double lifting effect

Double (2 thread) Lifting effect with one insertion!
21G cannula and 210mm/285mm thread can be applied to thin or sensitive areas.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	21G	60	LA	2-0	210	12
	21G	90		2-0	285	

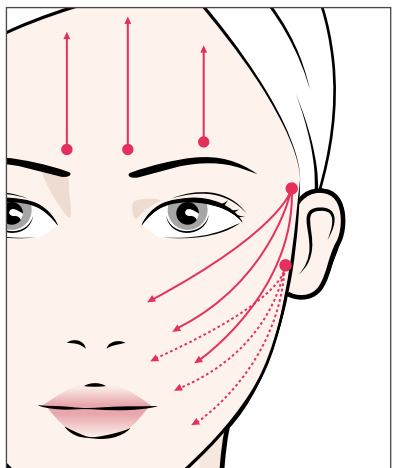

- 1/ After a punching, Insert into the SMAS.
- 2/ Insert them to the desired point.
- 3/ Remove cannula while pressing thread in the skin by hand.
- 4/ Lift up properly.
- 5/ Balance and cut the thread.

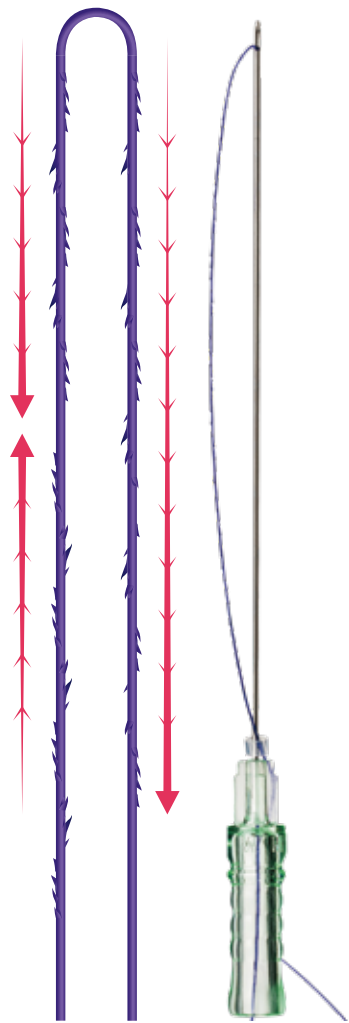


U-Cog




Wide Area Lifting

The area can be lifted with a double arm blunt needle.
Various applications from forehead, face to body lifting such as chest, hips.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	21G	110	Blunt	2-0	470	2
PCL	21G	180		2-0	470	

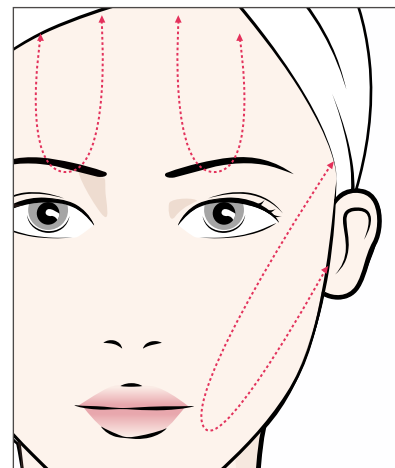

* There are various treatment methods.
* Please check our website video and contact us if you have any questions.
www.whitemedience.com

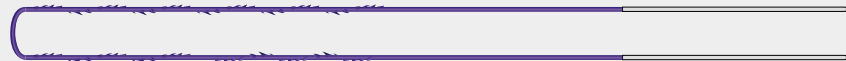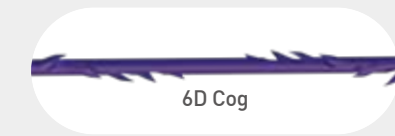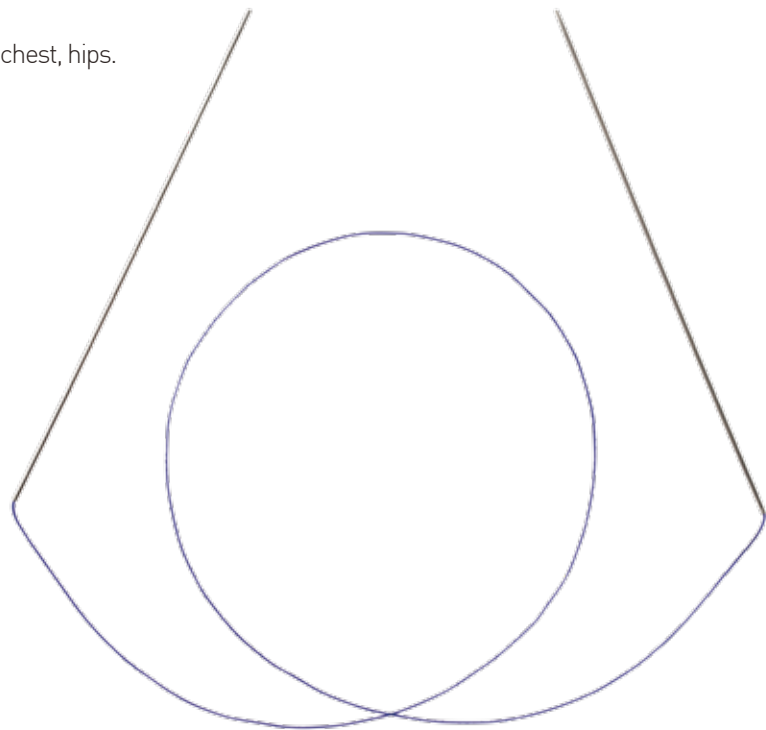


Hiko

Rhinoplasty

USP 3&4 size and the special press treatment maximize volume and supporting power for nose.
LA type(New L type) is not only easy to use, but also safer with more blunter tip than existing L type.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	19G	38	LA	3&4	45	4
PCL	19G	60		3&4	80	

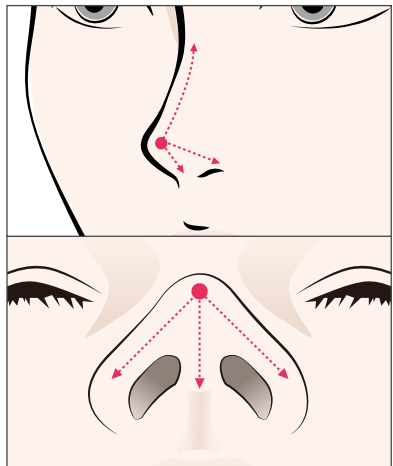

- 1/ After a punching, Insert into the SMAS.
- 2/ Insert them to the desired point.
- 3/ Remove cannula while pressing thread in the skin by hand.
- 4/ Lift up properly.
- 5/ Balance and cut the thread.

☆☆ SUPER SIZE Cog

USP
3&4





New Misko




Rhinoplasty

WHITE MEDIENCE EXCLUSIVE!
Disposable injector and unique thread make evolution in rhinoplasty

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	19G	50	0	1	26	4
	19G	50		1	40	

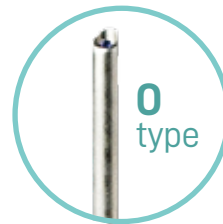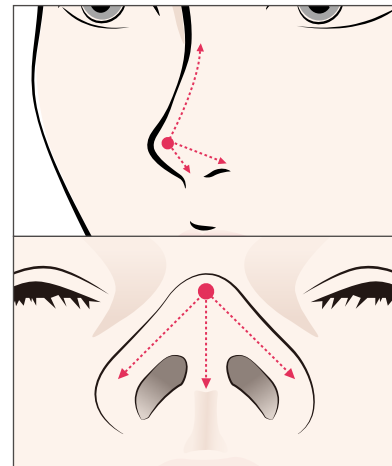

- 1/ Punch one entry point at the tip.
- 2/ Insert 2 into columella, 1 into both wing each.
- 3/ Insert the product while holding your nose with your fingers
- 4/ Insert it while making sure that it does not break through the lips
- 5/ Press the injector about 0.5cm so that the thread comes out slightly.
- 6/ If the thread gets caught in the flesh, lift the injector and remove the product

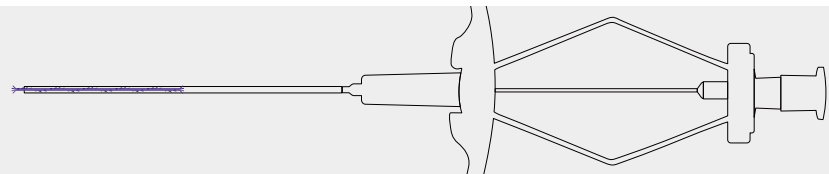


Volume

Mesh Thread

Mesh & Pipe. Braided 8 threads are good for volumizing and filling in lines and wrinkles.
* It can be used with Fillers.
LA type(New L type) is not only easy to use,
but also safer with more blunter tip than existing L type.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	18G	60	LA	5-0X8	60	6
PCL	19G	60		6-0X8	60	

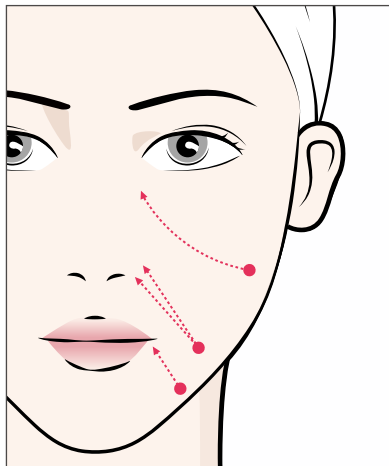

- 1/ Punch one entry point.
- 2/ Insert along wrinkle lines (1 entry point)
- 3/ Insert the product with the thread facing the skin
- 4/ Depending on the patient, Insert 1~3 pieces with 1 entry point
- 5/ Insert the thread fully.
- 6/ Remove cannula.




Duo Ten

18 Threads effect

Bunch of threads will maximize collagen producing. Totally 10 threads folded in half.
* It can be used with Fillers
LA type(New L type) is not only easy to use,
but also safer with more blunter tip than existing L type.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	21G	38	LA	5-0X10	50	6
PCL	21G	60		5-0X10	90	

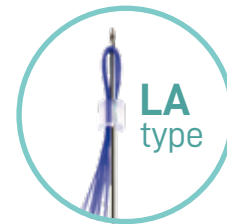

- 1/ Punch one entry point.
- 2/ Insert the product with the thread facing the skin
- 3/ According to the procedure design, Insert proper pieces.
- 4/ Insert fully and remove fixing pink handle at the end.
- 5/ Remove cannula. pulling 1 of the fixing threads out of the skin.




V-Roll


Cavern Screw

The final evolution of the cavern screw category.
The easiest and most effective in cavern type threads.
LA type(New L type) is not only easy to use,
but also safer with more blunter tip than existing L type.


	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	23G	60	LA	3-0	350	6
PCL	23G	60		4-0	410	

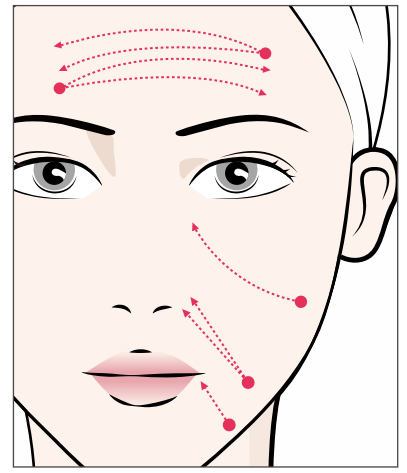

- 1/ Punch entry point.
- 2/ Insert the product while holding the thread with the auxiliary tube.
- 3/ Pulling out cannula by pressing the auxiliary tube toward the skin to prevent the thread from coming out.
- 4/ Remove the auxiliary tube after push thread once at for all

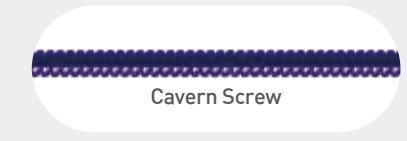

Cavern Screw


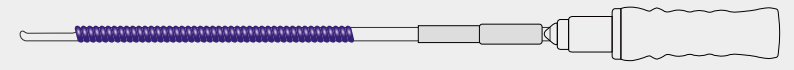


i-Filler

Twister Thread

Thin blunt cannula minimize bleeding and bruising.
Twister thread remove wrinkles by muscle relaxation and Collagen production.
The best for sensitive areas like around eyes.


	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	29G	25	N	6-0	35	10
	30G	25		7-0	35	
PCL	29G	25		5-0	35	
	30G	25		6-0	35	

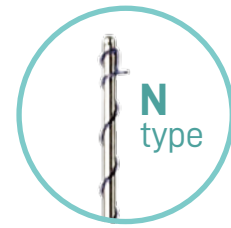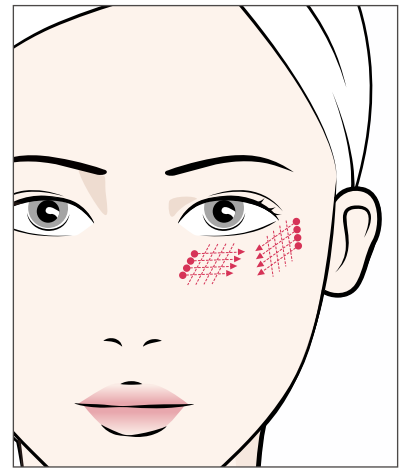

- 1/ Punch and insert the product.
- 2-1/ Insert along wrinkle lines
- 2-2/ Insert where you want to make voluming
- 3/ Depending on the patient, use proper quantities.
- 4/ Remove cannula.



Twister Thread


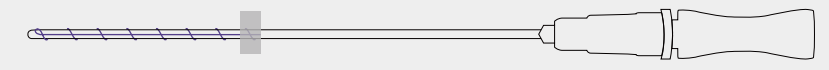


Mono


Special Needle & Extension

It's a very easy and short way to improve your skin.
Less pain and quick return to daily life.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	29G	25	S	6-0	35	50
	29G	38		6-0	50	
	27G	60		5-0	90	
PCL	29G	25	S	6-0	35	20
	29G	38		6-0	50	

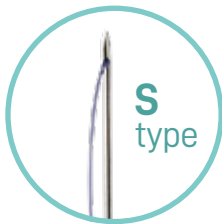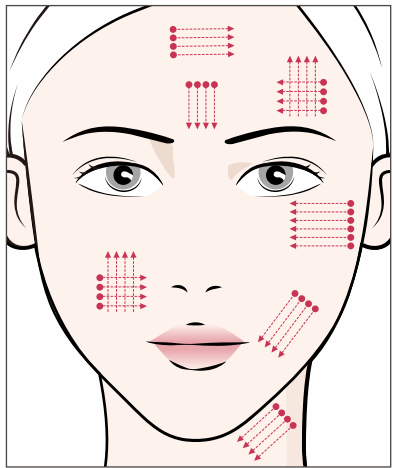

- 1/ Insert the product.
- 2/ Put densely all over the desired area.
* Normally 100~200 pieces on face
- 3/ Pull out the needle


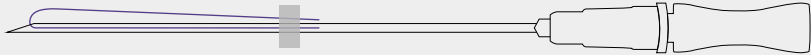


Screw

Special Needle & Extension

It's a very easy and short way to improve your skin.
Less pain and quick return to daily life.
Twisted thread can make more collagens than mono.

	Needle			Thread		PCS
	Gauge	Length	Type	USP	Length	
PDO	29G	25	S	6-0	35	50
	29G	38		6-0	50	
	27G	60		5-0	90	
PCL	29G	25	S	6-0	35	20
	29G	38		6-0	50	

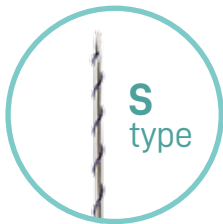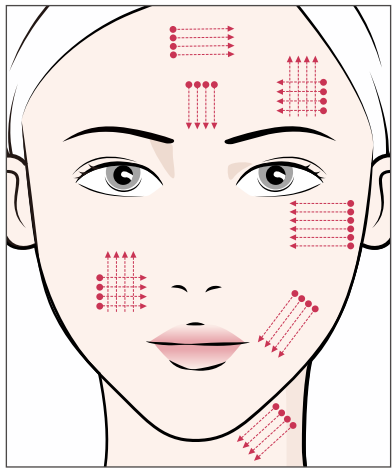

- 1/ Insert the product.
- 2/ Put densely all over the desired area.
* Normally 100~200 pieces on face
- 3/ Pull out the needle

